


How to Assemble

4 Place a bedsheet on top of the conduction mattress

3 Place the Conduction Mattress on top of savE® FS-29

2 Place three units of charged savE® FS-29 in the cradle

1 Place the cradle in the bassinet of the warmer

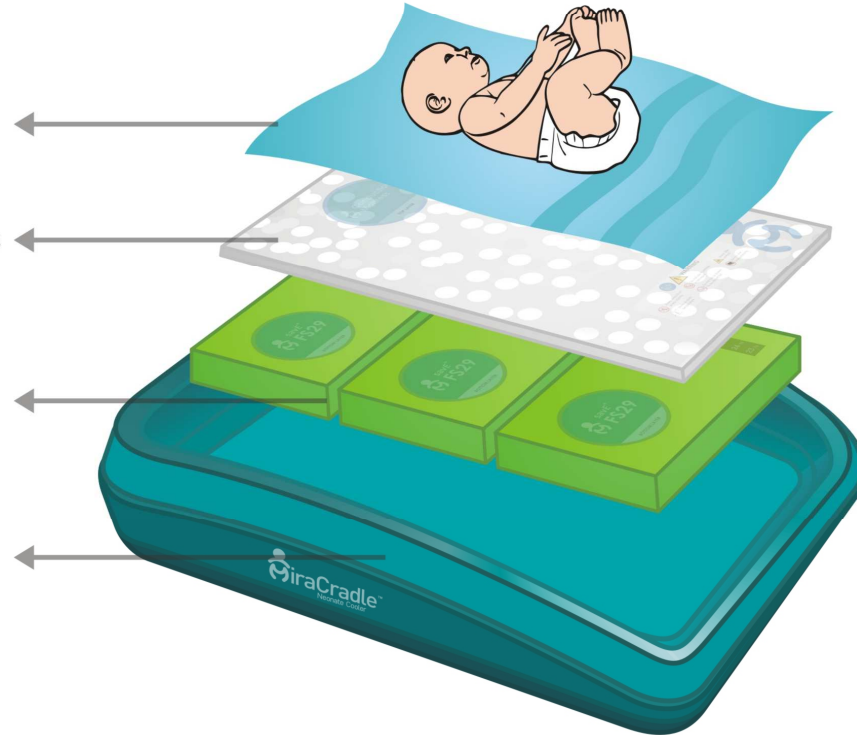

Important Checks before use



savE® FS-29 and savE® FS-21 should feel hard and rigid before use.



The components of MiraCradle™ – Neonate Cooler should be cleaned before use



Ensure that the temperature on the temperature indicator is between 23-26°C



Temperature alarm limits on the multiparameter monitor should be 33.2°C and 33.8°C



Ensure conduction mattress is flat and uniform before use



Read User manual carefully before use